

Innocor® Packing Instructions



Screw off the gas cylinder. Do NOT place the cylinder in the Innocor box or STARTKIT box.
Should you need to ship the cylinder, please use separate packaging.



Screw the white plug in in order to keep the area clean and dry.
If you have not kept the plug we sent, cover with tape.



Shield the Innocor with a plastic bag to protect it from dust.



Place foam 1 on a flat surface as shown in the picture.



Secure the Innocor as shown in the picture.



Place foam 2 on the Innocor as shown in the picture.



View from top with foam 2 in place.



Make sure the INNOCOR box is safely taped at the bottom.



Place the Innocor secured between the foams inside the INNOCOR box, as shown in the picture.



Pack the RVU, NIBP and pulse oximeter inside the STARTKIT box. Use bubble plastic, foam rubber or a plastic bag with foam to keep the items from rattling.



Close the STARTKIT box.



Place the STARTKIT box on the top of the Innocor in the large INNOCOR box. The STARTKIT box should ALWAYS be used, even if empty, to prevent the Innocor with foam inserts from moving freely inside the box.



Close the box, secure with tape.